

DISTAL TIBIAL AND PILON FRACTURES WITH THE RADIOLUCENT ANKLE CLAMP

Operative Technique

by DR. J.L. MARSH DR. F. LAVINI

- 7 -



CONTENTS

	Page n°
Introduction	1
Use of the RadioLucent Ankle Clamp	4
Equipment Required	
Cleaning and Sterilization	
Operative Technique	
Post-Operative Management	14
Use of the RadioLucent Ankle Clamp in conjunction with the 10000 Series Fixator	16
Use of the Hybrid Fixation System	16
Use of the Metaphyseal Clamp or the T-Clamp	17
References	18

INTRODUCTION

Fractures of the distal tibial metaphysis, with extension into the articular region, or pilon fractures, are commonly due to vertical compression trauma, or, more rarely, to torsional forces which result in a spiral fracture of the distal tibia with extension into the ankle joint.

It is clearly imperative to assess both the severity of joint involvement and the degree of metaphyseal comminution in order to determine the appropriate treatment strategy. Anatomic reconstruction of the articular surface is mandatory, but the degree of associated metaphyseal comminution will often mean that adequate support for the articular surface cannot be achieved without the introduction of an autologous bone graft.

While Type I fractures (articular fractures without significant displacement)¹ may in the majority of cases be reconstructed with minimal internal fixation, Type II fractures (articular fractures with significant disruption of the articular surface but minimal metaphyseal comminution), and Type III fractures (severely comminuted fractures with major joint involvement) require more extensive surgical intervention.

The recommended surgical treatment for these fractures has for many years been open reduction coupled with rigid internal fixation with plates and screws^{1,2,3,4,5}.

There is, however, a high incidence of complications associated with this method of treatment, and reports of wound breakdown, superficial and deep infection and osteomyelitis are common^{2,6,7,8,9,10,11,12}. Amputation is a significant risk in the more severe cases¹³.

Recently, several authors^{3,9,14,15,16} have reported decreased complication rates in severe pilon fractures using external fixation.

In view of the need to obtain accurate reconstruction of the articular surface in these circumstances, with minimal additional insult to the metaphyseal region where the soft tissue envelope is thin, limited open reduction of the articular fracture and fixation of the fragments with screws or Kirschner wires is indicated. Limited internal fixation may also be performed using the Orthofix Fragment Fixation System which comprises a range of implants of varying thickness and thread length specifically designed for this purpose, which can be introduced percutaneously and subsequently trimmed to length.

Use of the Orthofix Dynamic Axial Fixator with the RadioLucent Ankle Clamp has many advantages in this situation. The ankle module is applied medially to screws inserted in the talus and calcaneum, while the body of the fixator is attached to screws inserted into the tibial shaft. The telescopic feature of the fixator body provides the distraction necessary for adequate visualization of the joint while reconstruction of the articular surface is carried out. Unobstructed views of the distal tibial metaphysis and ankle joint can be obtained in any plane because of the radiolucency of the ankle clamp.

Reduction of the fragments in the metaphyseal region and the provision of adequate stability to maintain the integrity of the newly reconstructed joint surface can then be achieved by ligamentotaxis (exerting traction on the joint capsule and associated ligaments), again mediated via the telescopic facility of the fixator.

The articulated clamp is locked during the early stages of fracture healing, but can be unlocked at the appropriate time to allow gentle mobilization of the ankle about the centre of rotation of the tibio-talar joint without loss of fracture reduction.

Saleh et al.¹⁷ reported a series of 12 patients with severe pilon fractures (Rüedi and Allgöwer Types II and III) treated with the Orthofix Dynamic Axial Fixator with articulated clamp for the ankle. All required open reduction and minimal internal fixation of the joint surface. The authors considered the technique to be a safe, minimally invasive form of treatment for a difficult fracture.

Bonar and Marsh¹⁸ reported their results in 21 patients with severe pilon fractures (Rüedi and Allgöwer Type II, 9; Type III, 12; 7 open fractures), treated with the Orthofix system, combined in 15 with limited internal fixation. Nineteen of the 21 cases healed. The two instances of non union were in cases where there was no attempt at articular reconstruction and no bone grafting. Despite the high degree of bone and soft tissue injury, there were no cases of wound infection, skin slough or osteomyelitis. The authors comment that the limited approach coupled with external fixation avoided the soft tissue stripping necessary to obtain rigid internal fixation.

More recently, Marsh et al.¹⁹ have described the results of a prospective study in 49 displaced plafond fractures in 48 patients. The Orthofix articulated clamp for the ankle was used in all patients, with minimal internal fixation to reconstruct the articular surface in 40 cases. Average duration of external fixation was 12 weeks, and all fractures healed. This study was conducted in four different centres. There were no cases of wound breakdown or osteomyelitis, and no amputations.

In a prospective study randomized to traditional open reduction and internal fixation, or external fixation with minimal internal fixation, Wyrsch et al.¹³ found that there were 28 additional operations, 6 wound breakdowns, 6 deep infections and 3 (16%) amputations in the 19 patients of the ORIF group, compared with 5 additional operations, no wound breakdowns, no deep infections and no amputations in the 20 external fixator patients, most of whom were treated with the Orthofix system. Furthermore, the functional assessment of the external fixator patients was significantly improved compared with the ORIF patients.

In a prospective study in Johannesburg, South Africa, Wisniewski and Radziejowski²⁰ treated 21 consecutive patients with tibial pilon fractures with the Orthofix system combined with limited internal fixation. There were no cases of wound breakdown, no deep infections and no amputations. All the fractures healed within 16 to 24 weeks.

In the papers quoted above, a total of 123 patients with tibial pilon fractures have been treated with external fixation in seven different trauma units, the vast majority with Orthofix, without any wound breakdown, deep infection or amputation.

In pilon fractures where the degree of comminution is less severe, bridging of the ankle joint may be avoided with use of the Orthofix Hybrid Fixator Assembly²¹.

This system can also be used in non-articular fractures of the distal tibia where there is insufficient space for the application of a Metaphyseal Clamp or T-clamp with Orthofix screws.



All the techniques and procedures described in this manual are illustrated with reference to the ProCallus Fixator (90000 Series). The Dynamic Axial Fixator (10000 Series) can also be used for this indication (see page 16). The body of the device can be combined with the RadioLucent Ankle Clamp (90046) to construct the assembly necessary for treating distal tibial and pilon fractures.



With this fixator, the male component (a) slides within a groove in the female component (b) for adjustment of body length. Movement between the male and female components of the fixator is prevented when the Central Body Locking Nut (c) is tightened.

When loosened, the Micromovement Locking Nut unlocks a mechanism which allows limited cyclic micromovement on weightbearing, but prevents collapse of the fracture.

Loosening the Central Body Locking Nut provides progressive loading of the fracture site, but should not be carried out until the fracture is stable enough for weightbearing.

USE OF THE RADIOLUCENT ANKLE CLAMP

EQUIPMENT REQUIRED



- a) ProCallus Fixator Short Model (90028).
- b) Radiolucent Ankle Clamp (90046).
- c) Radiolucent Ankle Pin Guide (11947). This together with the RadioLucent Ankle Clamp is also available in a sterile kit (99-90646)*.
- d) Four or five 6/5 mm cortical screws. Screw length and thread length can be estimated by reference to the patient's X-rays, using the Orthofix transparent X-ray overlay. Thread length should be such that about 5 mm of thread will remain outside the entry cortex and about 2 mm will project beyond the second cortex.

The use of HA-Coated Screws is strongly recommended for this application.

- e) Two Kirschner Wires, 1.5 mm diameter, 250 mm long (11014).
- f) Three Kirschner Wires, 2.0 mm diameter, 150 mm long (11146).
- g) One Cannulated Drill Bit, 3.2 mm diameter, 150 mm long (1101301). Cannulation 1.8 mm.
- h) One Drill Bit, 4.8 mm diameter, 180 mm long (1100101).
- i) One Drill Guide, 3.2 mm diameter, 40 mm long (11106).
- j) One Drill Guide, 4.8 mm diameter, 40 mm long (11104).
- k) Three Screw Guides, 60 mm long (11102).
- I) One T-Wrench (11000).
- m) One Allen Wrench, 6 mm (10017).
- n) One Torque Wrench (10025).
- \triangle See instructions prior to use

STERILE EO

* CAUTION: Contents sterile unless package is opened or damaged; Do not use if package is opened or damaged.

CLEANING AND STERILIZATION

Unless sterile, when products are used for the first time, they should be removed from their containers and properly cleaned using a medical grade solution of alcohol in distilled water, minimum strength 70%.

Detergents with free fluoride, chloride, bromide, iodide or hydroxyl ions must not be used, as they will damage the black anodised coating on any Orthofix products.

After cleaning, the devices should be rinsed with sterile distilled water and dried using clean non-woven fabric.

Prior to surgical use, the fixator, bone screws and instrumentation should be cleaned as described above and sterilized by steam autoclaving following a validated sterilization procedure, utilizing a prevacuum cycle [Orthofix recommends the following cycle: steam autoclave 132°-135°C (270°-275°F), minimum holding time 10 minutes].

The fixator frame can be sterilized in the assembled state as long as the ball-joints, micromovement locking nut, central body locking nut, clamp cover screws and the articulation locking nut on the RadioLucent Ankle Clamp are left untightened.

Exposure of the fixator to high temperatures when any locking nuts or screws are tightened may cause breakage due to thermal expansion. It is also important for all the components to be loose to allow for adequate sterilization.

Please refer to Manual 1, "General Application Instructions" for more information on equipment maintenance.

OPERATIVE TECHNIQUE



The distal screws are inserted first, one in the talus and one in the calcaneum.

Screw localization is always in the centre of the neck of the talus, and in the upper part of the posterior aspect of the tuberosity of the calcaneum. The screws must be inserted perpendicular to the axis of the tibia, when viewed in both the coronal and sagittal planes.

The use of OsteoTite (HA-Coated) Bone Screws is strongly recommended in this application, especially in the talus and calcaneum.



Insertion of the Talar Screw

The first screw to be inserted is the talar screw. This screw may be inserted free-hand. A 1.5 mm Kirschner wire is inserted exactly in the position shown in the inset diagram, in the centre of the neck of the talus, in a line parallel to the talar dome.



Once the position of this Kirschner wire has been confirmed using image intensification, a screw guide with a 3.2 mm drill guide is inserted over the Kirschner wire and the bone drilled with the 3.2 mm cannulated drill bit, held exactly parallel to the top of the dome of the talus. The Kirschner wire and drill guide are replaced with a 4.8 mm drill guide, and the entry cortex enlarged with a 4.8 mm drill bit.

The talar screw is inserted in the usual manner, using image intensification to confirm penetration of the second cortex.



An and

The screw guide is now removed and the radiolucent pin guide applied over the talar screw. The screw guide is then re-applied. The handle of the pin guide should be aligned close to the axis of the tibia, to ensure correct screw positioning for a later full range of ankle mobilization.



The pin guide is centred over the approximate centre of rotation of the tibio-talar joint by aligning the radio-opaque circle of the pin guide with the centre of the circle described by the dome of the talus.





Alternative Technique

As an alternative to free-hand placement, the talar screw may be inserted using the pin guide as follows:

The exact position of the sinus tarsi is identified on the lateral aspect of the foot by palpation with the index finger. The thumb is then used to identify the projection of the sinus tarsi on the medial side of the foot.

The pin guide is now centred over the medial projection of the sinus tarsi and a 2 mm Kirschner wire inserted through the centre of the guide, down to the skin, so that it is parallel to the dome of the talus in the AP projection.

The handle of the pin guide is aligned close to the axis of the tibia, to ensure correct screw positioning for a full range of later ankle mobilization. Accurate positioning of the distal screws is aided by the fact that the pin guide is radiolucent, allowing unobstructed views on image intensification.

A 2 mm Kirschner wire is inserted into each of the two small holes in the pin guide, parallel to the dome of the talus, to stabilize the pin guide in the correct position.





A screw guide and a 3.2 mm drill guide guide are inserted into the anterior screw seat of the pin guide through a skin incision, after separation of the underlying soft tissues down to the bone.

The talar screw is then inserted using the same procedure described above.

Insertion of the Calcaneal Screw

The calcaneal screw is now inserted.

Placement of the calcaneal screw in the superior aspect of the calcaneum allows some dorsi-flexion of the ankle at the appropriate time in the post-operative phase (see page 14). If the handle of the pin guide is held in line with the axis of the tibia, this screw will normally be in the correct position.

A screw guide and 3.2 mm drill guide are inserted into the posterior screw seat and the 3.2 mm drill bit used to drill the bone, parallel to the top of the dome of the talus. The proximal cortex is enlarged with a 4.8 mm drill bit, as for

The proximal cortex is enlarged with a 4.8 mm drill bit, as for the talar screw.



The drill bit and drill guide are now removed and the calcaneal screw inserted.

Correct positioning of the distal screws should be checked by image intensification, using an AP view of the lateral border of the talus for the talar screw, and an axial view of the hindfoot for the calcaneal screw.

Insertion of the Diaphyseal Screws

The pin guide is removed together with the screw guides, and the fixator with radiolucent ankle clamp fitted, ensuring that the telescopic body is open by about 1.0 cm. When applying the ProCallus, the Micromovement Locking Nut should be TIGHTENED, and the Central Body Locking Nut LOOSENED.

Application sites for two or three cortical screws are now established in the medial face of the tibia, and these screws should now be inserted at right angles to the long axis of the tibia. Screws should be in positions 1 and 5 (two screws) or 1, 3 and 5 (three screws).

It is not necessary to use a separate template for insertion of the diaphyseal screws. This is because the ProCallus straight clamp allows for the insertion of screw guides.

Screw guides are inserted into the clamp and a 4.8 mm drill guide and drill bit used to drill the bone.



Once all the screws have been inserted, the screw guides are removed, the clamp cover screws tightened with the Allen wrench and reduction obtained.

Accurate reduction is aided by the fact that the ankle clamp is radiolucent, allowing unobstructed views on image intensification.

The ball-joints are then tightened using the Allen wrench. Final locking of the ball-joints is performed with the torque wrench.

The fixator is now distracted by 4 to 5 mm under image intensification, using the compression-distraction unit. Once distraction of the tibio-talar joint has been achieved, the Central Body Locking Nut should also be TIGHTENED. This distraction produces ligamentotaxis at the ankle joint, and will have the meduae here forements with each time.

and will help to reduce bone fragments with soft tissue attachments. Because the ankle clamp is radiolucent, full Image Intensifier views of the fracture site are available in all planes.





Visualization of the articular surface for limited internal fixation may be aided by temporarily increasing the distraction.

Bone grafting is often necessary for metaphyseal defects and is accomplished through small incisions.

In some cases internal fixation of the fibula through a separate incision may be indicated, particularly if the fibula fracture extends into the ankle joint, or if there is a tibio-fibular diastasis.



The foot is now placed in the neutral position and the articulation locking nut on the ankle clamp tightened.

POST-OPERATIVE MANAGEMENT

The condition of the soft tissues is of major importance in this injury. Where extensive damage to the soft tissues has occurred, elevation of the foot is often required in the post-operative phase until the situation improves.

Pin Site Care

The visible parts of the screws and surrounding skin should be cleaned on the day following application of the fixator assembly and at least once a day thereafter. Only sterile water should be used for this purpose. A dry absorbent dressing with additional gauze is used around the pin sites. After a few days, when they are dry, no dressing is needed.

There may be some loss of serous fluid, especially in overweight patients. This should not be mistaken for infection and is not a true complication. It may be the result of excessive patient mobility and subsequent irritation of the tissues around the screws. Normal care on pin cleaning is required.

Where inflammation is seen and the exudate is purulent, with the skin around the screw red and warm, a bacteriological swab should be taken and the appropriate antibiotic given for 7 to 10 days. Weightbearing should be restricted until resolution has occurred.

Should local conditions not improve, the patient should return to hospital for more aggressive therapy, including possible removal of the screw or screws involved.

Radiological Assessment

X-rays should be taken immediately post-operatively and every 3-4 weeks thereafter, throughout the healing period, to assess progress.

Weightbearing and Dynamization

Mobilization of the knee joint and exercises to strengthen the quadriceps and hamstring muscles should begin within the first one to two days following the operation.

Weightbearing on the affected limb is not permitted until the early signs of consolidation are observed. This may be as early as one month in simple cases, but may be delayed for 8-12 weeks in more complicated cases.

In the period before dynamization is instituted, the patient may be permitted to perform gentle flexion/extension exercises of the tibio-talar joint by loosening the articulation locking nut on the ankle clamp.

Full range of movement exercises are commenced as soon as the soft tissues are sufficiently healed.

In some patients, loosening of the articulation locking nut may result in the foot adopting an equinus position. Should this occur, it is preferable to unlock the ankle clamp only when physiotherapy is administered and/or for Continuous Passive Motion, keeping it locked, with the foot in the neutral position, at all other times.

Once there is clinical and radiological evidence of early fracture consolidation, the fixator should be dynamized by unlocking the central body locking nut, and weightbearing gradually increased. Perimalleolar edema may occur during this period. This should not give rise to concern since it is a common occurrence in lesions of this type and is always associated with application of the fixator in the region of the tibio-talar joint.

Occasionally, following a period of initial weightbearing, particularly when dynamization has been instituted too early, a certain degree of fracture collapse may occur. This may sometimes require slight adjustment of distraction.

Fixator Removal

Fracture healing is assessed on both clinical and radiological grounds. In most cases, the fixator can be removed at three months.

POSSIBLE COMPLICATIONS

Pin Site Problems

Very occasionally, early osteolysis and loosening may occur, particularly in association with the distal screws.

Distal screw loosening is common after 4 months, but fracture consolidation has usually occurred by this time.

These problems will be minimized if HA-Coated Bone Screws have been used.

Loss of Fracture Position

This may occur if the patient has not followed instructions regarding the time of onset of weightbearing or the gradual progressive increase in weightbearing recommended.

RECOMMENDED ACTIONS

If clinical and X-ray evidence suggest that this has occurred, the frame may need to be removed and a plaster cast applied to the foot, ankle and distal tibia, after an initial treatment period of 4-6 weeks.

In some instances this may be corrected by closed manipulation, but in others, depending upon the time elapsed since the fracture, and at the discretion of the surgeon, internal fixation may be applied. This complication is extremely rare provided the instructions regarding weightbearing are followed meticulously, since the frame is particularly stable when reduction is maintained under distraction.



USE OF THE RADIOLUCENT ANKLE CLAMP IN CONJUNCTION WITH THE 10000 SERIES FIXATOR

When using the 10000 Series Fixator, the assembly is prepared by removing the distal straight clamp (10006) and replacing it with the RadioLucent Ankle Clamp (90046). Use of a template is required for screw insertion in the diaphysis. The distal screws are inserted as described on pages 6-11. If the complete fixator is applied to the distal screws, and the fixator body opened by 1.0 cm, the positions for the diaphyseal screws can be marked on the skin. A standard clamp template (11107) is then used for screw insertion.

EXTRA-ARTICULAR APPLICATION USING THE HYBRID FIXATION SYSTEM

In pilon fractures where the degree of comminution is less severe, bridging of the ankle joint may be avoided with use of the Hybrid Fixation System. The ring carrying the tensioned wires can be coupled either with the Orthofix Dynamic Axial Fixator (10000 or 90000 Series) to form a Hybrid Fixator Assembly (A) or with a second ring and a Sheffield Clamp to form the Sheffield Hybrid Fixator Assembly (B).

This system can also be used in non-articular fractures of the distal tibia where there is insufficient space for the application of a Metaphyseal Clamp, a Torbay-Garches Clamp or a T-Clamp with Orthofix screws.

For details of the method of application of the Hybrid Fixation System, see Manual 12, parts A and B.



EXTRA-ARTICULAR APPLICATION USING THE MONOLATERAL SYSTEM

In fractures of the distal tibial metaphysis, where there is adequate room for the introduction of screws, or in pilon fractures with lesser degrees of comminution, the Orthofix Dynamic Axial Fixator (10000 or 90000 Series) with a straight clamp proximally and either a Metaphyseal Clamp, a Torbay-Garches Clamp or a T-Clamp distally may be used. Minimal internal fixation of the articular fragments to restore joint congruity may also be required and the Orthofix Fragment Fixation System has been designed for this purpose.

The Metaphyseal Clamp has a horizontal component for the introduction of screws at right angles to the long axis of the bone, and a vertical component which can be rotated to ensure optimal placement of an additional screw in line with the diaphyseal axis.

When this clamp is used, the first screw to be inserted is the more posterior of the distal screws, immediately anterior to the posterior border of the medial malleolus, parallel to the articular surface. The anterior distal screw is then inserted. The vertical component of the Metaphyseal Clamp template is then rotated to determine the most favourable point for insertion of the remaining metaphyseal screw.

REFERENCES

- 1) RÜEDI T., ALLGÖWER M.
- Fractures of the lower end of the tibia into the ankle-joint. Injury, 1969; 1: 92-99.
- AYENI J.P.
 Pilon fractures of the tibia: a study based on 19 cases. Injury, 1988; 19(2): 109-114.
- 3) MAST J.W., SPIEGEL P.G., PAPPAS J.N. Fractures of the tibial pilon. Clin. Orthop., 1988; 230: 68-82.
- 4) RÜEDI T.

Fractures of the lower end of the tibia into the ankle joint: results 9 years after open reduction and internal fixation. Injury, 1973; 5: 130-134.

- RÜEDI T., ALLGÖWER M. The operative treatment of intra-articular fractures of the lower end of the tibia. Clin. Orthop., 1979; 138: 105-110.
- 6) BOURNE R.B. Pilon fractures of the distal tibia. Clin. Orthop., 1989; 240: 42-46.
- 7) BOURNE R.B., RORABEC C.H., MACNAB J. Intra-articular fractures of the distal tibia: the pilon fractures. J. Trauma, 1983, 23: 591-595.
- DILLIN L., SLABAUGH P. Delayed wound healing, infection, and nonunion following open reduction and internal fixation of tibial plafond fractures. J. Trauma, 1986; 26 (12): 1116-1119.
- KELLAM J.F., WADDELL J.P. Fractures of the distal tibial metaphysis with intra-articular extension - the distal tibial explosion fracture. J. Trauma, 1979; 19: 593-601.
- OVADIA D.N., BEALS R.K. Fractures of the tibial plafond. J. Bone Joint Surg., 1986; 68A: 543-551.
- 11) PIERCE F.O. Jr., HEINRICH J.H. Comminuted intra-articular fractures of the distal radius. J. Trauma, 1979; 19: 828-832.
- TEENY S., WISS D.A., HATHAWAY R., SARMIENTO A. Tibial plafond fractures: errors, complication, and pitfalls in operative treatment. Orthop. Trans., 1990; 14 (2): 265.
- 13) WYRSCH B., McFERRAN M.A., McANDREW M., LIMBIRD T.J., HARPER M.C., JOHNSON K.D., SCHWARTZ H.S. Operative Treatment of Fractures of the Tibial Plafond. J. Bone Joint Surg., 1996; 78-A (11): 1646-1657.
- 14) BONE L., STEGEMANN P., McNAMARA K., SEIBEL R. The use of external fixation in severe fractures about the ankle. Orthop. Trans., 1990; 14 (2): 265.
- 15) RIES M.D., MEINHARD B.P. Medial external fixation with lateral plate internal fixation in metaphyseal tibia fractures: a report of eight cases associated with severe soft-tissue injury. Clin. Orthop., 1990; 256: 215-224.
- 16) BONE L.B. Fractures of the tibial plafond: the pilon fracture. Orthop. Clin. North Am., 1987; 18: 95-104.
- SALEH M., SHANAHAN M.D.G., FERN E.D. Intra-articular fractures of the distal tibia: surgical management by limited internal fixation and articulated distraction. Injury, 1993; 24 (1): 37-40.
- BONAR S.K., MARSH J.L. Unilateral External Fixation for Severe Pilon Fractures. Foot & Ankle, 1993; 14 (2): 57-64.
- MARSH J.L., BONAR S., NEPOLA J.V., DECOSTER T.A., HURWITZ S.R. Use of an Articulated External Fixator for Fractures of the Tibial Plafond. J. Bone Joint Surg., 1995; 77-A: 1498-1509.
- WISNIEWSKI T.F., RADZIEJOWSKI M.J. Combined Internal and External Fixation in Treatment of Pilon Fractures. SA Bone and Joint Surg. 1996; VI(2): 12-21.
- 21) BONIVENTO G.

Experience with a peri-articular attachment to the Dynamic Axial Fixator: a preliminary report. Supplement to International Journal of Orthopaedic Trauma, 1993; 3(3): 52-54.

Orthofix Operative Technique Manuals

- 1 GENERAL APPLICATION INSTRUCTIONS
- 2 GROWTH PLATE DISTRACTION
 - Chondrodiatasis
 - Hemichondrodiatasis
- 3 LIMB LENGTHENING AND CORRECTION OF DEFORMITIES BY CALLUS DISTRACTION
 - Callotasis
 - Hemicallotasis
 - Tibial lengthening and angular
 - correction with the OF-Garches limb lengthener
- 4 ARTHRODIATASIS (Articulated Joint Distraction)
 - Hip
 - Ankle
- 5 ARTHRODESIS (Joint Fusion)
 - Shoulder
 - Hip
 - Knee
 - Ankle
- 6 DIAPHYSEAL FRACTURES
 - Humerus
 - Forearm
 - Femur
 - Tibia
- 7 DISTAL TIBIAL AND PILON FRACTURES WITH THE RADIOLUCENT ANKLE CLAMP
- 8 PELVIC APPLICATIONS
- 9 TREATMENT OF FRACTURES AND DEFORMITIES IN SMALL BONES
- 10 THE PENNIG DYNAMIC WRIST FIXATOR
- 11 THE LIMB RECONSTRUCTION SYSTEM - Part A: General Principles
 - Part B: Correction of Deformities
- 12 THE HYBRID FIXATION SYSTEM
 - Part A: The Hybrid Fixator Assembly
 - Part B: The Sheffield Fixator Assembly

Orthofix Srl wishes to thank the surgeons listed below for their invaluable help in the preparation of this manual:

J.L. MARSH, MD

Department of Orthopaedic Surgery University of Iowa Hospitals and Clinics Iowa City, Iowa - USA

F. LAVINI, MD Clinica Ortopedica e Traumatologica Università degli Studi di Verona - Italy

Your Distributor is:

www.orthofix.com

