




VERO NAIL Cephalic Screwdriver - 17963



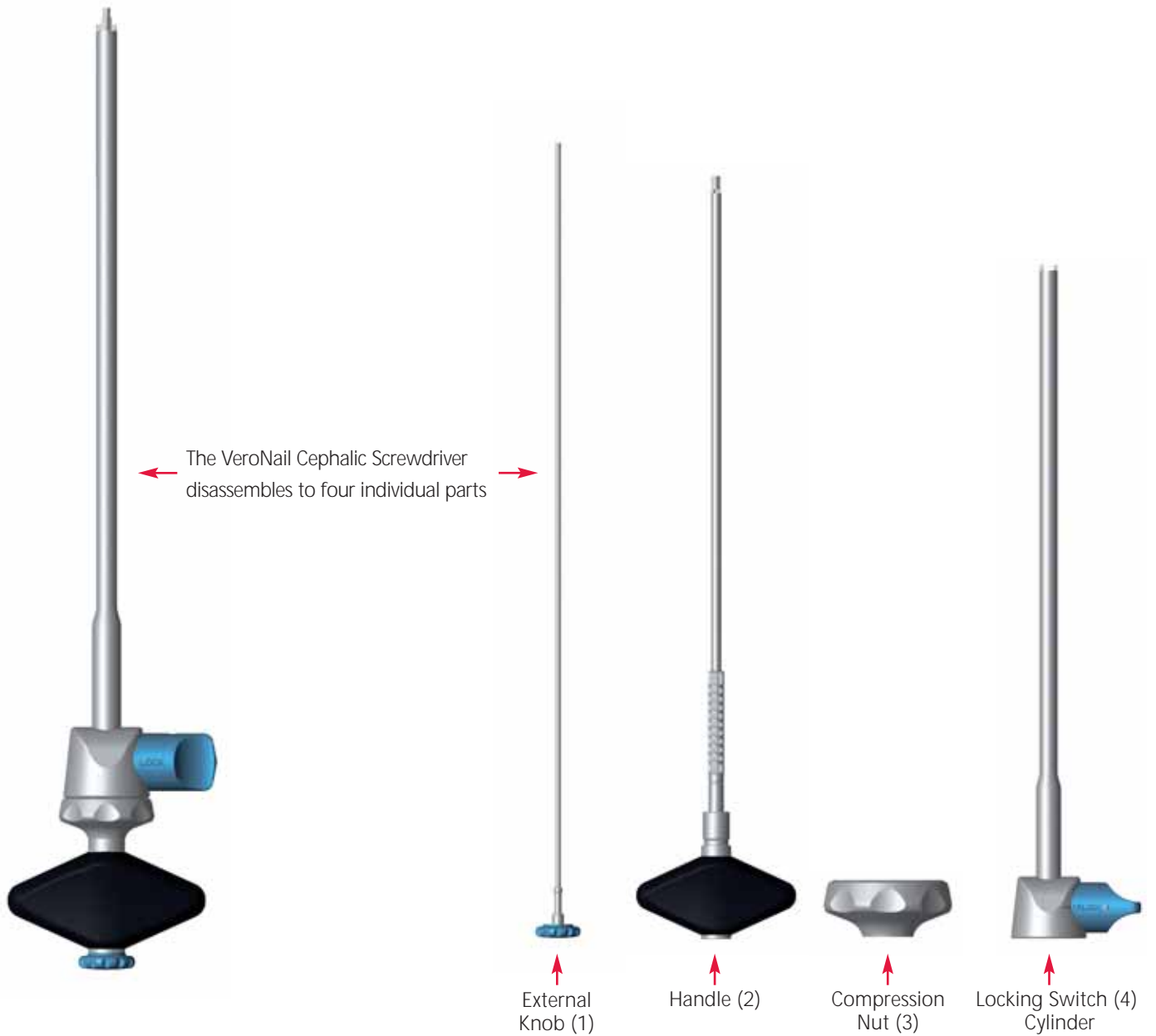

Disassembly

1. Turn the External Knob (1) in anti-clockwise direction to separate the hexagonal rod from the cannulated component.



2. Loosen Locking Switch (4) turning it in anti-clockwise direction.



3. Pull to remove the shaft from the Handle (2).



4. Turn the Compression Nut (3) clockwise and remove it from the assembly.



Assembly

1. Turn the Compression Nut (3) anti-clockwise onto the handle assembly (2).




2. Slide the shaft onto the handle assembly (2).



3. Turn the Locking Switch (4) in a clockwise direction to lock the cylinder.



4. Insert the External Knob (1) into the cannulated component and turn clockwise to connect it. The External Knob will freely rotate when connected.

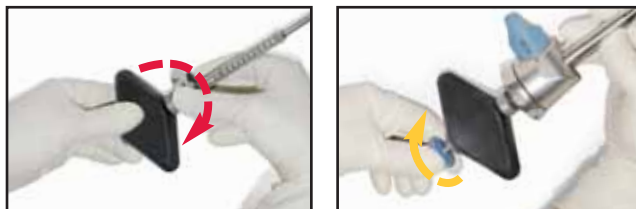


For cleaning, disinfection and sterilisation, please refer to instructions for use PQ ISP.

Manufactured by: ORTHOFIX Srl
Via Delle Nazioni 9
37012 Bussolengo (Verona)
Italy

Telephone +39-0456719000
Fax +39-0456719380



Your Distributor is:

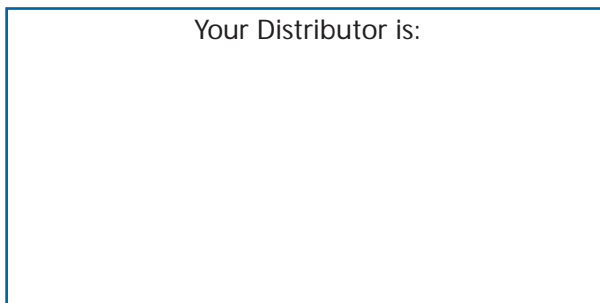

DISASSEMBLY | ASSEMBLY