 Orthofix Inc.
3451 Plano Parkway
Lewisville, Texas 75056-9453 U.S.A.
1-214-937-3199
1-888-298-5700
www.orthofix.com

Australian Sponsor
Emergo Australia
Level 20, Tower II
Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia

 Medical Device Safety Service
(MDSS)
Schiffgraben 41
30175 Hannover
Germany
www.mdss.com

Device System Name:
Manual Surgical Instruments, Orthopedic

 Click directory below for desired language

English	en	2-3
Deutsch	de	4-5
Français	fr	6-7
Italiano	it	8-9
Español	es	10-11
Português	pt	12-13

 Orthofix Inc.
 3451 Plano Parkway
 Lewisville, Texas 75056-9453 U.S.A.
 1-214-937-3199
 1-888-298-5700
 www.orthofix.com

Australian Sponsor
 Emergo Australia
 Level 20, Tower II
 Darling Park
 201 Sussex Street
 Sydney, NSW 2000
 Australia

 Medical Device Safety Service
 (MDSS)
 Schiffgraben 41
 30175 Hannover
 Germany
 www.mdss.com

English en

Device System Name:

Manual Surgical Instruments, Orthopedic

Description:

The following instructions are for special request Class I surgical instruments supplied by Orthofix. These instruments are manual surgical instruments.

Intended Use:

These surgical instruments are intended for use in orthopedic surgical procedures only by persons with specific training in the use of these surgical instruments. These are technically demanding procedures presenting a risk of serious injury to the patient.

Warnings and Precautions:

1. The use of a surgical instrument for tasks other than those for which they are intended may result in damaged or broken instruments or patient injury.
2. Make sure the product is used only by qualified or trained staff as these instruments are used in technically demanding procedures presenting a risk of serious injury to the patient.
3. The surgical instrument must be cleaned and sterilized before each surgical procedure. Always inspect instruments between uses for visual defects and to confirm proper functioning.
4. Do not use instruments that are broken, discolored, corroded, have cracked components, pits, gouges, or are otherwise found defective.
5. Improper use, handling or reprocessing of surgical instruments may result in damaged or broken instruments.
6. When reprocessing surgical instruments, always handle with care, wearing personal protective equipment such as clothing, gloves and eyewear in accordance with local health and safety procedures.
7. Instruments should be processed separately to avoid electrolytic action between the metals resulting in pitting and rusting of stainless steel instruments.
8. Delicate surgical instruments require special handling to prevent damage to the tips. Use caution during cleaning and sterilization. A soft cloth or soft plastic brush should be used to clean off all blood and debris.
9. Do not apply excessive stress or strain at joints; misuse will result in misalignment or cracks.
10. Aluminum and titanium instruments that are color anodized may lose their color over time through normal use and reprocessing.
11. The end of useful life for metal surgical instruments is normally determined by wear and accumulated damage due to the intended surgical use. Do not reuse single use devices.

For additional information on indications for use, warnings, precautions and potential adverse events please refer to the system "Instructions for Use" supplied with the instrument/implant set.

Cleaning:

Instruments are provided clean but not sterile.

All instruments must be thoroughly cleaned prior to and after each use. Cleaning may be done using validated hospital methods or following the validated cleaning process described below.

Additional instructions required for disassembly of the specific instrument are provided with the product when applicable.

From Point of Use:

Whenever possible, do not allow blood, debris or body fluids to dry on instruments. For best results and to prolong the life of the surgical instrument reprocess immediately after use.

1. Remove excess body fluids and tissue from instruments with a disposable, non-shedding wipe. Place instruments in a basin of purified water or in a tray covered with damp towels. Do not allow saline, blood, body fluids, tissue, bone fragments or other organic debris to dry on instruments prior to cleaning.
2. For optimal results, instruments should be cleaned within 30 minutes of use or after removal from solution to minimize the potential for drying prior to cleaning.
3. Used instruments must be transported to the central supply in closed or covered containers to prevent unnecessary contamination risk.

Note: Soaking in proteolytic enzymatic detergents or other pre-cleaning solutions facilitates cleaning, especially in instruments with complex features and hard-to-reach areas (e.g. cannulated and tubular designs, etc.). These enzymatic detergents as well as enzymatic foam sprays break down protein matter and prevent blood and protein based materials from drying on instruments. Manufacturer's instructions for preparation and use of these solutions should be explicitly followed.

Preparation for Cleaning:

1. All instruments with moving parts (e.g., knobs, triggers, hinges) should be placed in the open position to allow access of the cleaning fluid to areas that are difficult to clean.
2. Soak the instruments for a minimum of 10 minutes in purified water prior to the manual or automated cleaning process.
3. Use a soft cloth or a soft plastic bristle brush to remove any visible soil from the instruments prior to manual or automated cleaning. Use a soft plastic bristle brush or a pipe cleaner to remove soil from any inner lumens. You can also use a syringe (if appropriate) for hard to reach areas.
4. Enzymatic detergent should be used for manual and automated cleaning. All enzymatic detergents should be prepared at the use dilution and temperature recommended by the manufacturer. Softened tap water may be used to prepare the enzymatic detergents. Use of recommended temperatures is important for optimal performance of enzymatic detergent.

Manual Cleaning:

1. Completely submerge instruments in an enzymatic detergent and allow to soak for 20 minutes. Use a soft-bristled, nylon brush to gently scrub the device until all visible soil has been removed. Particular attention must be given to crevices, lumens, mated surfaces, connectors and other hard-to-clean areas. Lumens should be cleaned with a long, narrow, soft-bristled brush (i.e. pipe cleaner brush).
2. Remove the instruments from the enzymatic detergent and rinse in tap water for a minimum of 3 minutes. Thoroughly and aggressively flush lumens, holes and other difficult to reach areas.
3. Place prepared cleaning solution in a sonication unit. Completely submerge device in cleaning solution and sonicate for 10 minutes.
4. Rinse instrument in purified water for at least 3 minutes or until there is no sign of blood or soil on the device or in the rinse stream. Thoroughly and aggressively flush lumens, holes and other difficult to reach areas.
5. Repeat the sonication and rinse steps above.
6. Remove excess moisture from the instrument with a clean, absorbent and non-shedding wipe.
7. Inspect the instruments for visible soil.
8. If visible soil is noted, repeat the steps listed above.

Automated Cleaning:

1. Completely submerge the instruments in an enzymatic detergent and allow to soak and sonicate for 10 minutes each. Use a soft nylon bristled brush to gently scrub the device until all visible soil has been removed. Particular attention must be given to crevices, lumens, mated surfaces, connectors and other hard to clean areas. Lumens should be cleaned with a long, narrow, soft nylon bristled brush (i.e. pipe cleaner). Use of a syringe or water jet will improve flushing of difficult to reach areas and closely mated surfaces.
2. Remove instruments from the cleaning solution and rinse in purified water for a minimum of 1 minute. Thoroughly and aggressively flush lumens, blind holes and other difficult to reach areas.
3. Place instruments in a suitable washer/disinfector basket and process through a standard instrument washer/disinfector cleaning cycle.
4. Orient instruments into the automated washer's carriers as recommended by the washer manufacturer.
5. The following minimum parameters are essential for thorough cleaning.
 - a. 2 minute prewash with cold tap water
 - b. 1 minute prewash with hot tap water
 - c. 2 minute detergent wash with hot tap water (64-66°C/146-150°F)
 - d. 1 minute hot tap water rinse
 - e. 2 minute thermal rinse with purified water (80-93°C/176-200°F)
 - f. 1 minute purified water rinse (64-66°C/146-150°F)
 - g. 7 to 30 minute hot air dry (116°C/240°F)
6. Inspect the instruments for visible soil.
7. If visible soil is noted, repeat the above listed steps until no visible soil is noted.

Note: Certain cleaning solutions such as those containing caustic soda, formalin, glutaraldehyde, bleach, and/or other alkaline cleaners may damage instruments. These solutions should not be used.

Note: Visually inspect instruments after cleaning and prior to each use. Discard or return to Orthofix any instruments that are broken, discolored, corroded, have cracked components, pits, gouges, or are otherwise found defective. Do not use defective instruments.

Clean instruments with a ratchet lock in the first ratchet position before sterilization to avoid temperature induced stress cracks in the box locks. If closing the ratchet all the way creates a tight space move the ratchet to at least the second position to aid in heat distribution.

Consider replacing any blunt, worn out, flaking, fractured, corroded, stained, discolored or damaged instruments.

Torque Limiting Instrument Maintenance

- If a torque-limiting handle has been dropped, impacted or used incorrectly, return to Orthofix.
- Torque-limiting handles require maintenance at minimum, every three years or per your service agreement. Please return your torque limiting handles to Orthofix for required maintenance.

Instrument End of Life Determination:

Do not reuse Single Use instruments. Visually inspect the reusable instruments to determine if the instrument has reached end of life. Orthofix reusable instruments have reached End of Life when:

1. Instruments show signs of damage such as binding, bending, breakage, overt signs of wear and/or any other conditions which may impact the devices safe and effective use.
2. Instruments intended for cutting bone and/or tissue (e.g. tap, rasp, curette, rongeur) – when any of the cutting surfaces show signs of wear such as nicks, abrasions or otherwise dulled cutting surfaces.
3. Instruments that interface with other devices (e.g. implants, instruments, handles) - when the mating feature binds, fails to attach or fails to hold the device securely. The instrument function should be verified prior to each use.
4. Do not use instruments which reached End of Life. Discard End of Life instruments per your hospital procedure or return to Orthofix for disposal.

Sterilization:

Instruments are supplied NON-STERILE. First, loosen the closed connections of each instrument as applicable (e.g., a threaded handle) to provide gaps for steam to pass. Then, individually wrap each instrument in a FDA cleared sterilization wrap and place in the autoclave for sterilization by the hospital using one of the following recommended cycles:

Method: Steam	or:	Method: Steam
Cycle: Gravity		Cycle: Prevac
Temperature: 270°F (132°C)		Temperature: 270°F (132°C)
Exposure time: 15 minutes		Exposure time: 4 minutes
Drying time: 30 minutes		Drying time: 30 minutes

Note: Special request instruments are not validated for sterilization in any system's instrument case.

Packaging:

Packages for each of the components should be intact upon receipt. Damaged packages or products should not be used and should be returned to Orthofix.

Individual instruments are provided in sealed poly bags with individual product labels.

Product Complaints:

Any Healthcare Professional (e.g., customer or user of these products) who has any complaints or who has experienced any dissatisfaction with the product quality, identity, durability, reliability, safety, effectiveness and/or performance, should notify Orthofix Inc., 3451 Plano Parkway, Lewisville, TX 75056, USA, by telephone at 1-214-937-3199 or 1-888-298-5700 or by email at complaints@orthofix.com.

Latex Information:

The instruments and/or packaging material for orthopedic manual surgical instruments are not formulated with and do not contain natural rubber. The term "natural rubber" includes natural rubber latex, dry natural rubber, and synthetic latex or synthetic rubber that contains natural rubber in its formulation.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rx Only		Federal (U.S.A.) law restricts this device to sale by or on the order of a physician	
	See Instructions for Use	REF	Catalogue Number
	Orthofix.com/IFU		Manufacturer
	Single Use Only Do Not Reuse	ECREP	Authorized Representative
	Provided Non-Sterile	SN	Serial Number
LOT	Lot Number		

 Orthofix Inc.
3451 Plano Parkway
Lewisville, Texas 75056-9453 U.S.A.
1-214-937-3199
1-888-298-5700
www.orthofix.com

Australian Sponsor
Emergo Australia
Level 20, Tower II
Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia

 Medical Device Safety Service
(MDSS)
Schiffgraben 41
30175 Hannover
Germany
www.mdss.com

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