

SpinalStim™

Spinal Fusion Therapy



SpinalStim Device Quick Instructions:

1. Press the **On/Off Button** to start.
2. The **LCD** will show your daily compliance percentage, then your prescribed daily treatment begins.
3. Wear the device over treatment site for a minimum of **2 hours** every day.*
4. After treatment is complete, the device will power off.
5. To recharge, plug the charger into **Charging Port** and AC outlet.

*Clinical trial data demonstrates that success rates were significantly higher for patients who wore the device every day for the full treatment time. *Source: Premarket Approval P850007/S6; FDA Approved February 1990.*
See the instruction manual for full usage information.

Orthofix products or services referenced herein are trademarks or registered trademarks of Orthofix Medical Inc. and its group of companies. All rights reserved.



[Orthofix.com](https://www.Orthofix.com)

[BoneGrowthTherapy.com](https://www.BoneGrowthTherapy.com)

P/N 20122282 Rev AD 2022-01-12
SS-2203 © Orthofix US LLC 01/2022



SpinalStim™

Spinal Fusion Therapy



El Dispositivo SpinalStim Instrucciones rápidas:

1. Pulse el **Botón de encendido/apagado** para comenzar.
2. La pantalla **LCD** le mostrará su porcentaje de cumplimiento diario; después comenzará su tratamiento diario prescrito.
3. Use el estimulador sobre el lugar de tratamiento durante un mínimo de **2 horas** todos los días.*
4. Después de completar el tratamiento, el dispositivo se apagará.
5. Para recargarlo, enchufe el cargador en el **Puerto de carga** y en un tomacorriente de pared de CA.

*Los datos de los ensayos clínicos demuestran que las tasas de éxito fueron considerablemente más altas en los pacientes que usaron el dispositivo todos los días durante el tiempo completo de tratamiento. Fuente: Aprobación premercado P850007/S6; aprobado por la FDA en febrero de 1990

Consulte el manual de instrucciones para la información completa de uso.

Orthofix products or services referenced herein are trademarks or registered trademarks of Orthofix Medical Inc. and its group of companies. All rights reserved.



Orthofix.com

BoneGrowthTherapy.com

P/N 20122282 Rev AD 2022-01-12
SS-2203 © Orthofix US LLC 02/2022

 **ORTHOFIX®**